

## **FOOD SENSITIVITY**

PROCESS OVERVIEW







Your bespoke supplement package



The Healthpath plate



The Healthpath fundamentals of health programme



Sleep



Activity



Stress



Diet



### **FOOD SENSITIVITY**

#### TEST REPORT

Thank you for taking the Food Sensitivity Test—we're delighted to provide your personalised report.

The report is divided into three sections:

#### Scientific overview

The first section gives a brief overview of food sensitivity testing. This can help you to better understand your results.

#### Your results

II.


Ш.

The second section contains your personal results, along with guidance on how to interpret them.

#### Recommendations

The third and final section contains diet and lifestyle recommendations that enable you to make the most of these test results.

# i. SCIENTIFIC OVERVIEW

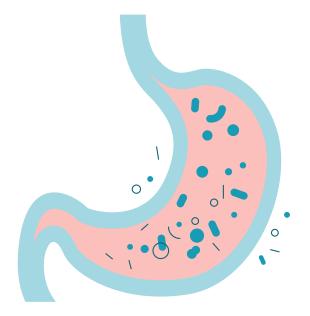

The purpose of digestion is to break food down into its smallest possible components e.g. proteins are broken down into amino acids. These small components pass through the gut wall into the bloodstream, and are then delivered to where they're needed in the body.

However, sometimes small fragments of partially digested or undigested food are able to pass through the gut wall. Your immune system recognises these as 'foreign' and responds by making IgG antibodies to them.

During a food sensitivity test, a sample of your blood is dispensed onto pads that contain food extracts. If IgG antibodies to that food are present in the blood, they bind to the food extract and create an antigen-antibody complex. These complexes can then be measured by a special high-resolution scanner.





#### II. YOUR RESULTS



What's key to understand is that this test does not diagnose food allergies. Food allergies involve another branch of your immune system (namely IgE antibodies) and generally necessitate life-long avoidance of the food in question. Food sensitivities are different. We're much more likely to become sensitive to a food if:



a) we eat it regularly

ana



b) our digestive function is impaired in some way

Food sensitivities don't necessarily mean you need to avoid a food forever.

For more advice on how to respond to these results, see the 'Your Recommendations' section below.



## III. RECOMMENDATIONS



It's important to understand that you can have high level of IgG antibodies without experiencing any symptoms. This may be because the immune system is efficient at clearing away the antigen-antibody complexes before they cause any problems. It's also important to appreciate that we're more likely to have high levels of IgG antibodies if our digestion is impaired in some way. If we're not breaking down food properly, or if we have leaky gut, it's more likely that larger fragments of food will reach the bloodstream.

With that in mind, it's essential to interpret these results in the context of your lifestyle and symptoms. These results are most useful when you use them as a springboard for further investigation. Let's look at two scenarios:



#### You have lots of foods in the red

As mentioned, we're more likely to be reactive to foods if our digestion isn't working properly. If you have lots of foods that have flagged up red or amber, it's most beneficial to pay attention to your gut health. Focus on the Healthpath Fundamentals of Health and eat according to the Healthpath Plate, monitoring your symptoms as you go. This will be more worthwhile (and more enjoyable) than simply eliminating lots of foods. If your symptoms haven't changed after 2–3 months of following the above, you may wish to do some further investigation. The Healthpath Gut Health MOT Test or Healthpath Leaky Gut Test are both good options.



#### You have surprising foods in the red

Remember, these results are only valuable when considered in the context of your life. If you've flagged up red for foods that you suspected were a problem, it's worth eliminating these foods entirely while you focus on supporting your gut health. Eat according to the Healthpath Plate, and consider trying the Healthpath Gut Restore supplement bundle. If you've flagged up red for a certain food but are certain that it doesn't contribute to your symptoms, then it's probably not worth eliminating it. Remember, not all IgG antibodies cause symptoms. However, if you're at all unsure, you can regard this as a valuable prompt for investigation. Eliminate that food entirely for three weeks, then follow the reintroduction process outlined below. It will become clear if that food is problematic for you.

#### REINTRODUCING FOODS

After three weeks, reintroduce one food at a time. Eat that food (e.g. eggs) two or three times in one day. Wait 48 hours to see if you have a reaction, recording anything you observe in the symptom diary below. If there's no reaction, you can once again include that food in your diet. You can then repeat this process with any other foods you have eliminated. If there is a reaction, you may wish to consider eliminating that food for a longer period while you focus on supporting your gut health.


Food	Eggs					
Date re-introduced	10/01/2018					
		Symptoms	observed within	the following 48	hours	
Digestive function	Bloating					
Joint/muscle aches						
Headaches	Mild headache on first day					
Nasal and/or lung congestion						
Bladder function						
Skin						
Energy/focus						
Sleep						
Other symptoms	Felt a bit down					

4


Disclaimer: if you're pregnant, breastfeeding, taking medications or suffering from a disease or medical condition, please consult your doctor before following these recommendations.

# Food IgG Test results are delivered in your private Healthpath dashboard

## Your practitioner considers your symptoms:

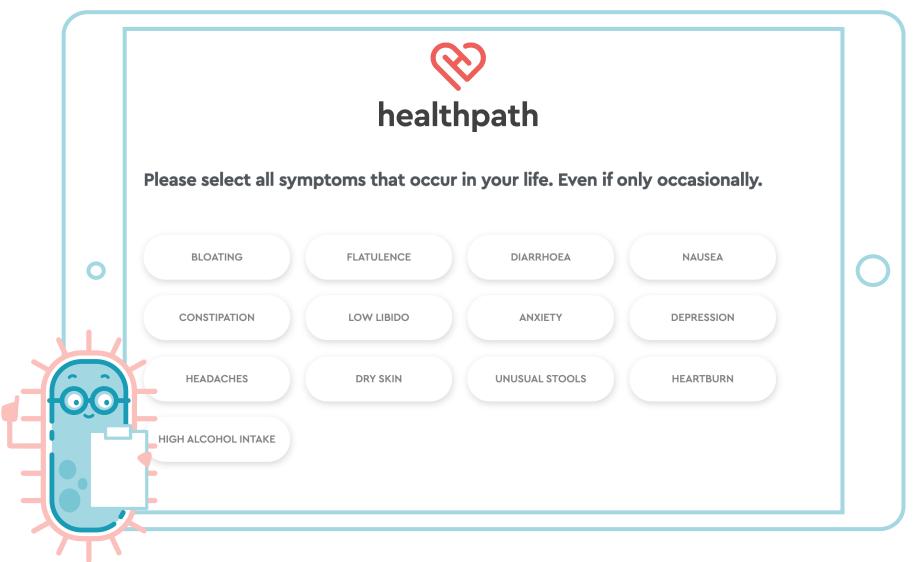



## You get a personal note from your practitioner:

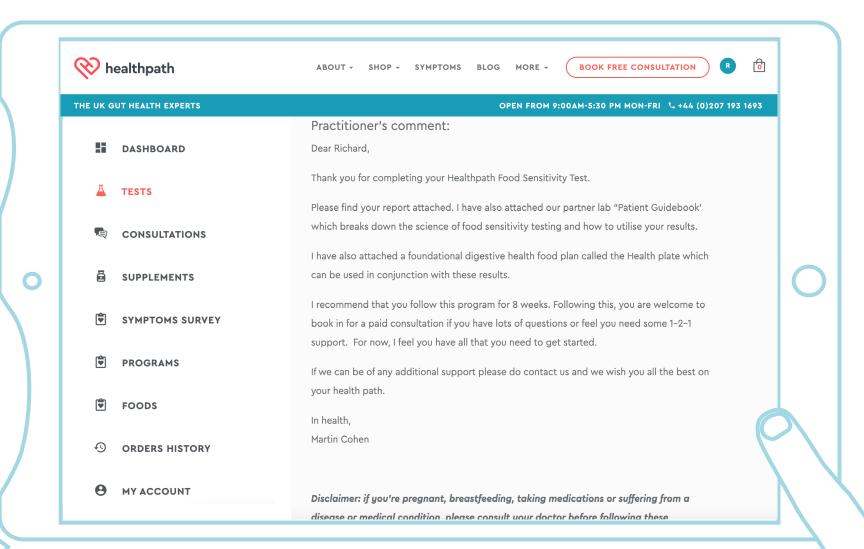

healthpath.com


## The fundamentals of health:

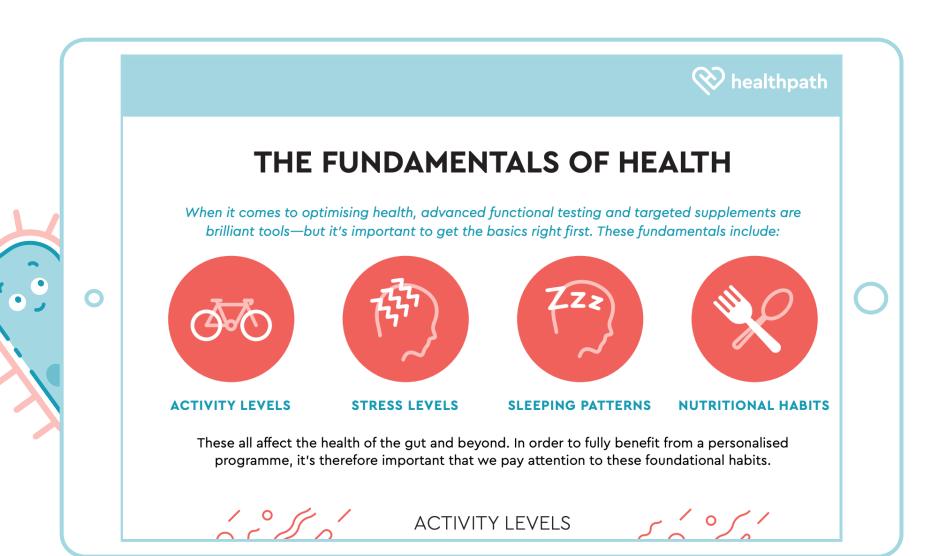

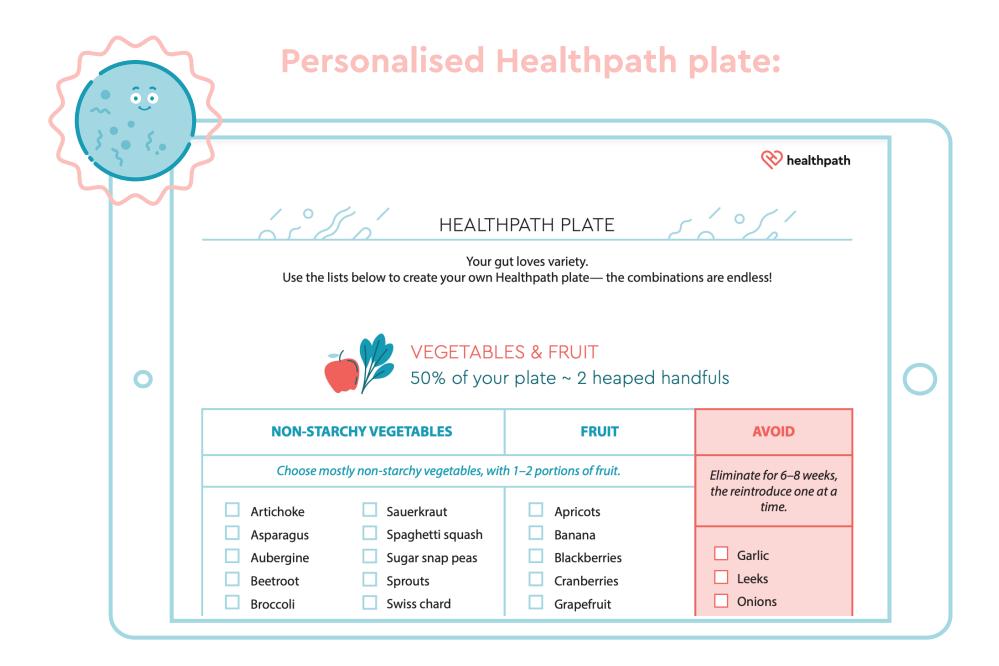



# Targeted supplement recommendations:

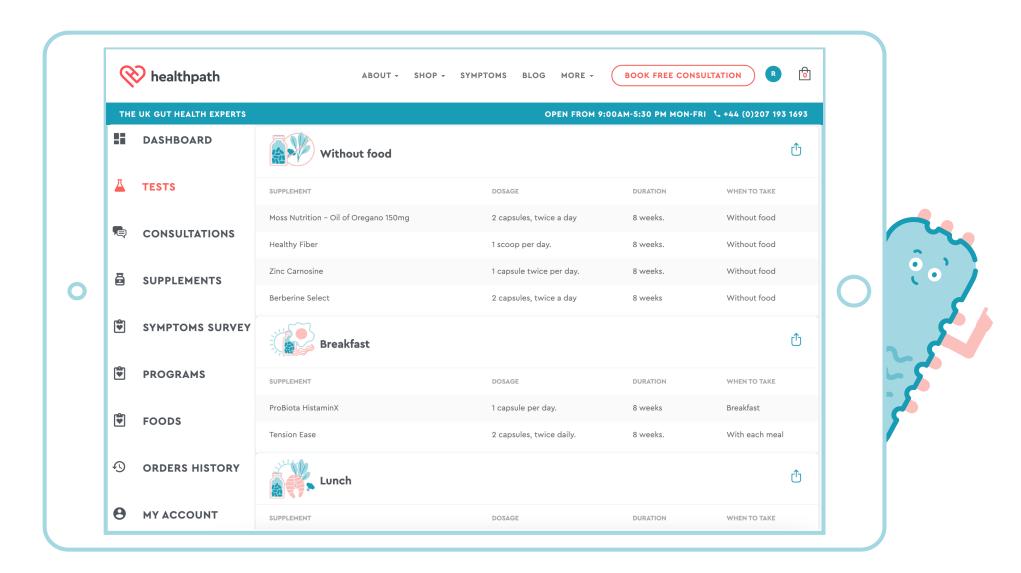

# Plus a copy of the original lab report

Seg White	ELEVATED (≥30 U/ml)		BORDERLINE (24-29 U/ml)		NORMAL (≤23 U/ml)
Barley	DAIRY / EGG				
Sample   S			, ,		
Sarley	23 Egg Yolk	63	Milk (Goat)		
20   Durum Wheat   30   Oat   50   Wheat	GRAINS (Gluten-Containing)*				
Section   Sect	54 Barley	26	Gliadin*	20	Rye
17	20 Durum Wheat	30	Oat	50	Wheat
Registration   Regi	GRAINS (Gluten-Free)				
Apple	24 Corn (Maize)	17	Rice		
B	FRUIT				
Blackberry	3 Apple	6	Grapefruit	8	Pineapple
Cherry	8 Avocado	13	Lemon	0	Raspberry
VEGETABLES	1 Blackberry	10	Lime	2	Strawberry
VEGETABLES   12   Bean (Red Kidney)   16   Cabbage (Savoyl/White)   20   Potato	5 Cherry	28	Orange		
12	7 Grape (Black/Red/White)	1	Pear		
Soya Bean   Soya Bean   Soya Bean   Soya Bean	VEGETABLES				
11	12 Bean (Red Kidney)	16	Cabbage (Savoy/White)	20	Potato
Pea   Pea   Pea   Pea	36 Bean (White Haricot)	4	Carrot	27	Soya Bean
Cod	11 Broccoli	2	Cauliflower		
1	9 Brussel Sprout	61	Pea		
1	FISH / SEAFOOD				
	1 Cod	16	Oyster	6	Trout
MEAT	1 Crab	4	Plaice	5	Tuna
MEAT					
0         Beef         0         Lamb         2         Turkey           2         Chicken         4         Pork         Turkey           HERBS / SPICES           5         Chilli (Red)         10         Ginger         0         Peppercorn (Black/White)           0         Garlic         24         Mustard Seed           NUTS / SEEDS           22         Almond         17         Cashew Nut         23         Peanut           20         Brazil Nut         27         Hazelnut         38         Pistachio           MISCELLANEOUS           17         Mushroom         23         Yeast (Baker's)         Yeast (Brewer's)           ***Ciadin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to gliadin, it is important to eliminate consumption of foods that	6 Lobster	5	Shrimp/Prawn		
Chicken	MEAT				
HERBS / SPICES	0 Beef	0	Lamb	2	Turkey
5         Chilli (Red)         10         Ginger         0         Peppercorn (Black/White)           0         Garlic         24         Mustard Seed           NUTS / SEEDS           22         Almond         17         Cashew Nut         23         Peanut           20         Brazil Nut         27         Hazelnut         38         Pistachio           MISCELLANEOUS           17         Mushroom         23         Yeast (Baker's)         Yeast (Brewer's)           * Gladin (gluten) is tested separately to the gluten-containing grains.         If your Test Report shows an elevated reaction to gliadin, it is important to eliminate consumption of foods that	2 Chicken	4	Pork		
O         Garlic         24         Mustard Seed           NUTS / SEEDS         22         Almond         17         Cashew Nut         23         Peanut           20         Brazil Nut         27         Hazelnut         38         Pistachio           MISCELLANEOUS           17         Mushroom         23         Yeast (Baker's)         Yeast (Brewer's)           ** Gliadin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to gliadin, it is important to eliminate consumption of foods that	HERBS / SPICES				
NUTS / SEEDS	5 Chilli (Red)	10	Ginger	0	Peppercorn (Black/White)
22 Almond 17 Cashew Nut 23 Peanut 20 Brazil Nut 27 Hazelnut 38 Pistachio  MISCELLANEOUS  17 Mushroom 23 Yeast (Baker's) Yeast (Brewer's)  *Glädin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to glädin, it is important to eliminate consumption of foods that	0 Garlic	24	Mustard Seed		
22 Almond 17 Cashew Nut 23 Peanut 20 Brazil Nut 27 Hazelnut 38 Pistachio  MISCELLANEOUS  17 Mushroom 23 Yeast (Baker's) Yeast (Brewer's)  *Glädin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to glädin, it is important to eliminate consumption of foods that	NUTS / SEEDS				
20 Brazil Nut 27 Hazelnut 38 Pistachio  MISCELLANEOUS  17 Mushroom 23 Yeast (Baker's) Yeast (Brewer's)  *Gliadin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to gliadin, it is important to eliminate consumption of foods that	22 Almond	17	Cashew Nut	23	Peanut
17 Mushroom 23 Yeast (Baker's) Yeast (Brewer's)  *Gladin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to gliadin, it is important to eliminate consumption of foods that	20 Brazil Nut		Hazelnut		Pistachio
*Gliadin (gluten) is tested separately to the gluten-containing grains. If your Test Report shows an elevated reaction to gliadin, it is important to eliminate consumption of foods that	MISCELLANEOUS				
	17 Mushroom	23	Yeast (Baker's)		Yeast (Brewer's)
					portant to eliminate consumption of foods that

